


Quick guide – Capturing intraoral images

Preparation

Planmeca Romexis® software

- 1 Select intraoral exposure from Planmeca Romexis 2D module.

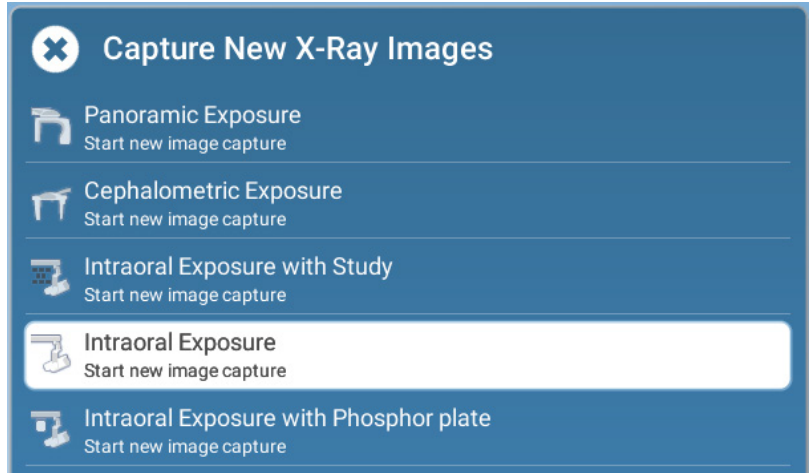


Planmeca ProSensor® HD

- 2 Select sensor size according to the type of the exposure and patient size.
- 3 Put the sensor into sensor sheath and use an appropriate sensor holder.



Patient and sensor positioning

- 1 Ask the patient to sit down.
- 2 Position the sensor into the patient's mouth.
- 3 Set the X-ray tube at an angle appropriate for exposure.

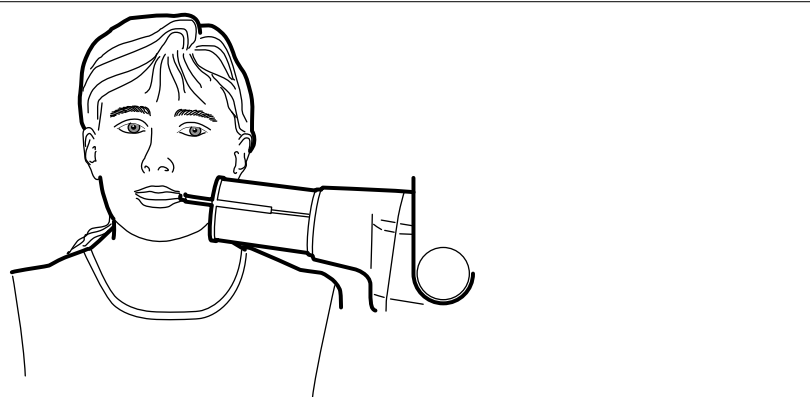


- 4 Select the appropriate exposure parameters from the X-ray unit.



Capture intraoral image

1 Press and hold down the exposure button.

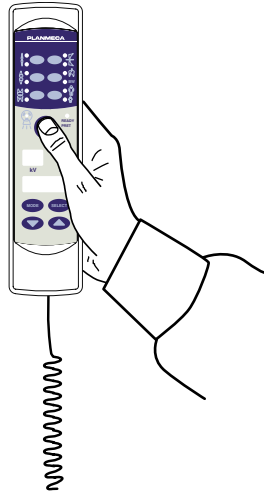

2 The image appears on computer screen.

3 Define the tooth numbers and sensor position.



4 Remove the sensor from patient's mouth when all exposures have been taken.